


THE  
**aarhus**  
SYSTEM™

Driver Assembly Instructions

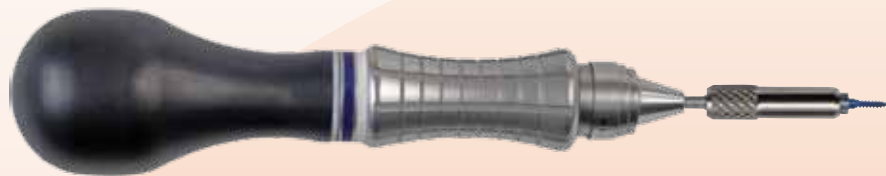

REV 6/11

# Driver Assembly Instructions

## Screwdriver Handle Options

75.23.78A



75.23.50



## Screw Release Sleeve

907.50.07



## Extended Sleeve

907.50.08



## Screwdriver Blade

75.23.39



## Extended Blade

75.23.91




## Assembled Blade and Sleeve



1. Slide Sleeve onto Blade

2. Twist to engage threads

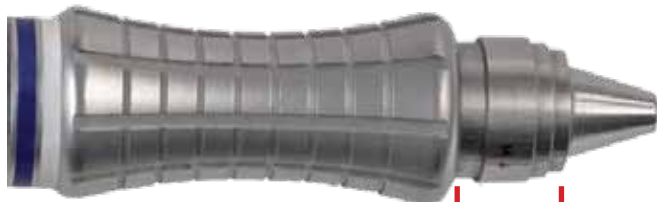

3. Retract and Hold



4. Slide assembly completely into handle end



5. Release to lock assembly in place