

The instructions contained in this document apply to the following part numbers:

861-181	SUPPLIES - MIRROR PHOTOGRAPHIC STAINLESS STEEL BUCCAL LATERAL
861-183	SUPPLIES - MIRROR PHOTOGRAPHIC STAINLESS STEEL CHILD/ADULT BUCCAL
861-185	SUPPLIES - MIRROR PHOTOGRAPHIC STAINLESS STEEL CHILD/ADULT OCCL SMALL
861-187	SUPPLIES - MIRROR PHOTOGRAPHIC STAINLESS STEEL CHILD/ADULT OCCL LARGE

Photographic Mirrors

REF 631-090A through 631-090G

Cleaning & Sterilization Instructions

The recommended methods are steam autoclave or dry heat sterilization.

INSTRUCTIONS FOR BOTH AUTOCLAVE AND DRY HEAT STERILIZATION:

- 1. Ultrasonic Cleaning:** Use standard ultrasonic cleaner and solution, taking care to prevent the mirror from contacting any other product or metal cassette tray.
- 2. Hand Wash:** Hand wash the mirror with non-abrasive material (soft tissue or gauze).
- 3. Sterilization:** After ultrasonic cleaning and hand washing the photographic mirrors, prior to sterilization, we strongly recommend to pat the mirror dry with a gauze sponge or lint free surgical towel. This will eliminate any water spots during sterilization. Wrap the mirror in a surgical towel, if you choose, when you are autoclaving. If you choose to dry heat sterilize the mirror you can wrap this product, making sure the mirror does not come in contact with any other products and/or metal cassette trays.



#631-090A



#631-090B



#631-090C



#631-090D



#631-090E



#631-090F



#631-090G

Note:

Chemical (Cold) Sterilants: In regards to chemical sterilants, there is such a large variety of this particular product in both the orthodontic and dental field. It is difficult for *Ortho Organizers* to recommend one particular cold sterilant for chemical sterilization. Cold sterilants in 90% of the procedures have proven to be acceptable. Keep in mind that problems do arise with cold sterilants, such as streaks or pitting on the mirror. If you choose to go with cold sterilization, we recommend you contact the manufacturer or experiment with only one mirror, to see the final results.

Photographic Mirrors

REF 631-090A through 631-090G

Cleaning & Sterilization Instructions

The recommended methods are steam autoclave or dry heat sterilization.

INSTRUCTIONS FOR BOTH AUTOCLAVE AND DRY HEAT STERILIZATION:

- 1. Ultrasonic Cleaning:** Use standard ultrasonic cleaner and solution, taking care to prevent the mirror from contacting any other product or metal cassette tray.
- 2. Hand Wash:** Hand wash the mirror with non-abrasive material (soft tissue or gauze).
- 3. Sterilization:** After ultrasonic cleaning and hand washing the photographic mirrors, prior to sterilization, we strongly recommend to pat the mirror dry with a gauze sponge or lint free surgical towel. This will eliminate any water spots during sterilization. Wrap the mirror in a surgical towel, if you choose, when you are autoclaving. If you choose to dry heat sterilize the mirror you can wrap this product, making sure the mirror does not come in contact with any other products and/or metal cassette trays.



#631-090A



#631-090B




#631-090C



#631-090D



#631-090E

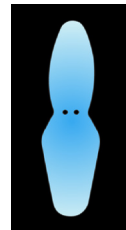

#631-090F



#631-090G

Note:

Chemical (Cold) Sterilants: In regards to chemical sterilants, there is such a large variety of this particular product in both the orthodontic and dental field. It is difficult for *Ortho Organizers* to recommend one particular cold sterilant for chemical sterilization. Cold sterilants in 90% of the procedures have proven to be acceptable. Keep in mind that problems do arise with cold sterilants, such as streaks or pitting on the mirror. If you choose to go with cold sterilization, we recommend you contact the manufacturer or experiment with only one mirror, to see the final results.



Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
USA
Tel: 800.547.2000, +(1) 760 448 8600
Fax: 800.888.7244, +(1) 760 448 8607
USASales@OrthoOrganizers.com
IntlOrders@OrthoOrganizers.com



mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen, Germany
49 511 39 08 9530



OMA Member
Orthodontic Manufacturers
Association

INS137A
Rev B

Visit the *Ortho Organizers'* Web site at OrthoOrganizers.com



Ortho Organizers, Inc.
1822 Aston Avenue
Carlsbad, CA 92008
USA
Tel: 800.547.2000, +(1) 760 448 8600
Fax: 800.888.7244, +(1) 760 448 8607
USASales@OrthoOrganizers.com
IntlOrders@OrthoOrganizers.com



mdi Europa GmbH
Langenhagener Str. 71
30855 Langenhagen, Germany
49 511 39 08 9530



OMA Member
Orthodontic Manufacturers
Association

INS137A
Rev B

Visit the *Ortho Organizers'* Web site at OrthoOrganizers.com