


BONDING INSTRUCTIONS

- Prepare teeth using standard office protocol.
- Firmly press adhesive into back of bracket.
- Seat on tooth with enough pressure to express perimeter adhesive “flash.”
- Remove adhesive flash and polymerize adhesive pursuant to your curing light instructions.
- Use a standard utility pliers to remove Visual Placement Aid (VPA) by its tail.



DEBONDING INSTRUCTIONS

Debonding Radiance Plus is simple and predictable thanks to specially designed ceramic bracket debonding instruments.

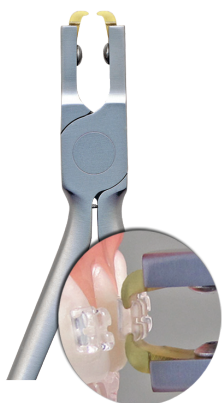

Using Radiance Plus Debonding Pliers

#001-343E

This instrument provides a unique option for debonding Radiance Plus brackets. The tips are designed to grasp each bracket under the tie wing rather than making contact at the enamel/adhesive interface. These pliers are pre-adjusted with set screws to ensure that an optimal amount of pressure is applied to the bracket when the handles are squeezed together.

- Remove any remaining adhesive flash not removed at initial bonding with an debonding bur.
- Place the opposing tips of pliers occlusal/gingival under the tie wings.
- Apply constant pressure to the handles of the pliers.
- Gently move the angle of the pliers to the occlusal or gingival of the tooth with increasing force until the Radiance Plus bracket releases from the tooth.

#001-408 Replacement tips (Qty 2 with wrench)

Tips are easily replaced and should be disinfected prior to use.



Using Omega Ceramic Bracket Debonding Pliers

#001-301E

Also known as Sushi pliers, this unique debonding instrument is specially designed for removing ceramic brackets. Only one edge of each blade is sharpened to ensure optimal fit along the enamel/adhesive interface. Opposing edges are parallel when the debonding instrument is open 3 mm, which is the average width of a ceramic bracket. Leave arch wires and ligatures in place during debonding.

- Remove any remaining adhesive flash not removed at initial bonding with a debonding bur.
- Place blades of recommended debonding instrument along the enamel/adhesive interface, occlusal/gingival for upper centrals, mesial/distal for all other brackets.
- Squeeze handles slowly until bracket releases from the tooth.



SCAN FOR




DEBONDING
DEMONSTRATION


Bracket Color Identification

Visual Placement Aids (VPA's) are color coded to each specific tooth.


A McLaughlin, Bennett, Trevisi* System

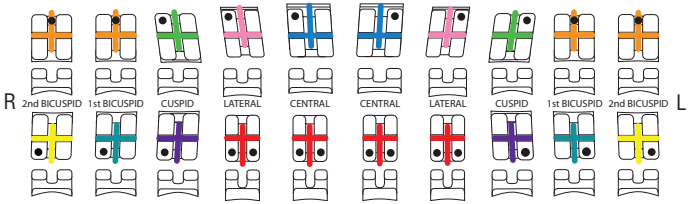

A Roth* System

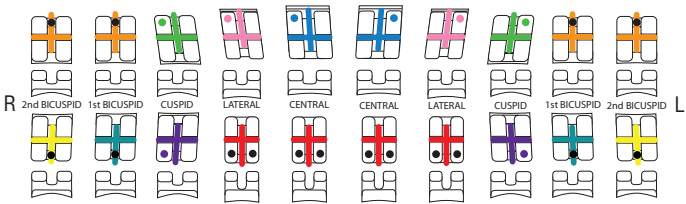

The Alexander* LTS System

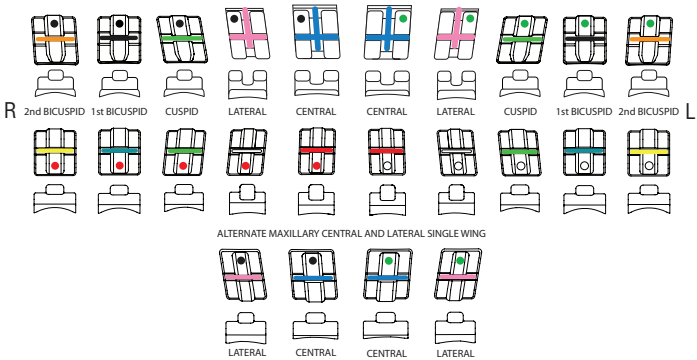

* The American Orthodontics version of the Roth System is not claimed to be a duplication of any other, nor does American Orthodontics imply that it is endorsed in any way by Dr. Roth.

* The American Orthodontics version of the McLaughlin, Bennett, Trevisi System is not claimed to be a duplication of any other, nor does American Orthodontics imply that it is endorsed in any way by Drs. McLaughlin, Bennett, or Trevisi.

* Dr. Wick Alexander is proud to endorse Alexander prescription brackets from American Orthodontics.



3524 Washington Avenue Sheboygan, WI USA 53081 920-457-5051 USA and Canada: 1-800-558-7687
 info@americanortho.com www.americanortho.com © 2013 American Orthodontics Corporation

Proudly made in the USA

LITCOS-09 REV3

For Instructions for Use go to www.americanortho.com

EC REP Emergo Europe • Molenstraat 15 • 2513 BH The Hague • The Netherlands

