

## BONDING TIPS

- Prepare teeth using standard office protocol.
- Firmly press adhesive into back of bracket.
- Seat on tooth with enough pressure to express perimeter adhesive “flash.”
- Remove adhesive flash and polymerize adhesive pursuant to your curing light instructions.
- Use a standard utility pliers to remove Visual Placement Aid (VPA) by its tail.



## DEBONDING


Debonding Radiance Plus is simple and predictable thanks to specially designed ceramic bracket debonding instruments.

### Using Ceramic Bracket (Sushi) Debonding Pliers

REF 001-017403

Only one edge of each blade is sharpened to ensure optimal fit along the enamel/adhesive interface. Opposing edges are parallel when the debonding instrument is open. Leave arch wires and ligatures in place during debonding.


- Remove any remaining adhesive flash not removed at initial bonding with a debonding bur.
- Place debonding blades along the enamel/adhesive interface.
- Squeeze handles slowly until bracket releases from the tooth.

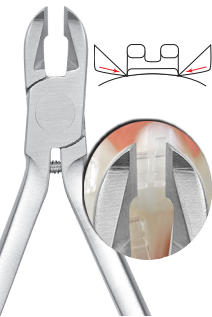

### Using LUNO Debond Tool

REF 001-0133040


This unique debonding instrument is designed for removing ceramic or aesthetic brackets. With a simple, intuitive mesial/distal rocking motion, the ceramic brackets easily debond.

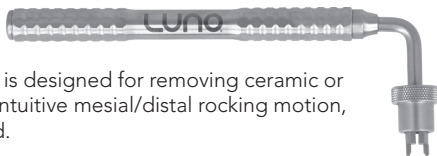


# Bracket Color Identification

All Radiance Plus Brackets come with Visual Placement Aids (VPA's) that are color coded to each specific tooth.

## McLaughlin, Bennett, Trevisi\* System Compatible

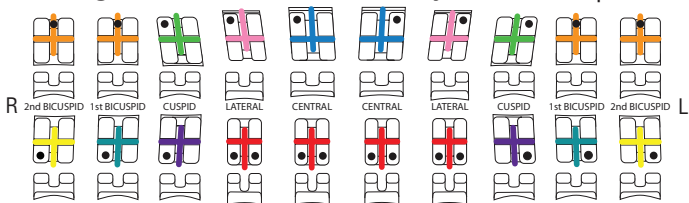

## Roth\* System Compatible

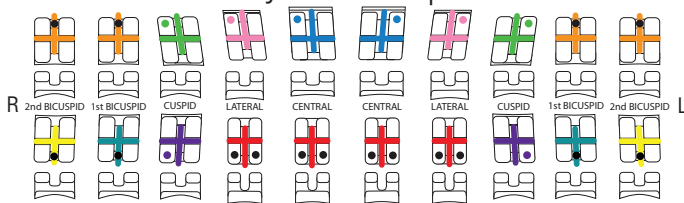

\* The American Orthodontics version of the Roth, McLaughlin, Bennett, Trevisi Systems do not claim to be a duplication of any other, nor does American Orthodontics imply that they are endorsed in any way by Drs. Roth, McLaughlin, Bennett, or Trevisi.



3524 Washington Avenue | Sheboygan, WI 53081 USA  
+1 920 457 5051 | [cs@americanortho.com](mailto:cs@americanortho.com) | [www.americanortho.com](http://www.americanortho.com)

©2021 American Orthodontics Corporation All Rights Reserved

Made in USA | FOLLOW US ON    